

MOIST HEAT EYE MASK

Air-Activated Eye Mask for Relief and Comfort

on the go!



A convenient, single-use moist heat eye mask designed by doctors, to relieve dry, tired eyes.

OPEN YOUR EYES
to Bruder
a Hilco Vision Company

HOW IT WORKS

Eyedration™ by Bruder® Air-Activated Moist Heat Eye Mask provides eye relief and comfort. It is convenient, fast acting and easy to use.



MOIST HEAT EYE MASK

Air-Activated for Eye Relief & Comfort

on the go!



Developed by
Eye Doctors



Helps Relieve
Dry, Tired Eyes



Fast Acting
Moist Heat Lasts
Up to 20 Minutes

What is an Eyedration mask? Eyedration is a soft, self-heating, single-use eye mask that, once opened, provides quick therapeutic, moist heat lasting up to 20 minutes, delivering relief and promoting natural eye hydration.

How does it heat up? It's air-activated; no microwave or cord needed. When the package is opened, the mask's natural ingredients automatically heat to the appropriate temperature needed to melt the meibum.

How can it help? When applied, the mask warms the glands in the eyelids releasing natural oils that help keep tears from evaporating, providing comfort for tired, irritated eyes.

Designed for effective dry eye relief at home or on the go. Individually-wrapped masks offer portability and convenience. Discard after use.

Active Ingredients: Iron powder, activated carbon, water, inorganic salt, vermiculite.



10 individually packaged masks per unit.

eyedration™
by Bruder®
a Hilco Vision Company



UNLOCK THE EYE'S NATURAL HYDRATION. Therapeutic moist heat provides fast acting relief. Simply open, place over eyes, for up to 20-minutes of the soothing moist heat.

How can it be used in office? The Eyedration Mask provides a clean, one-time-use solution for preparing patients prior to in-office procedures such as manual expression or eyelid debridement. Clearing obstructed meibomian glands and restoring the flow of healthy lipids into the tear film is an essential step for patients with ocular surface disease or meibomian gland disease (MGD). Use of the Eyedration mask can enhance your results, while introducing the benefits of moist heat treatments to your patients. Read more at bruder.com/pro/eyedration



Interested in carrying dry eye products in your practice?

Contact Bruder at eyes@bruder.com or 888-827-8337. | www.bruder.com/pro

Bruder®
a Hilco Vision Company

B88800-V2310