

KEY BENEFITS



- Safely and effectively cleans the eyelid margin, removing debris such as mascara, makeup residue, and bacterial biofilm. This encourages a clean, healthy lid margin environment and is also helpful in maintaining the hygiene of those with eyelash extensions.
- Smooth flexible micro-rubbed tips designed specifically for daily use.
- Made from a non-toxic, soft silicone material.
- Re-usable, compact, portable and easy to clean.
- Each iLidClean® cleaning tool comes in a convenient plastic container.
- Doctor designed and tested.
- Made in the USA.

Learn about our complete line of products designed to address your eyes' essential needs



HYGIENE

- Bruder® Hygienic Eyelid Sheets
- Bruder® Hygienic Eyelid Solution
- Bruder® Hygienic Eyelid Cleansing Wipes with and without Tea Tree Oil



HEAT

- Bruder® Moist Heat Eye Compresses
- Eydration® Air-Activated Moist Heat Eye Compress
- Eyeleve® Contact Lens Compress
- Samureye® Game Enhancing Mask



HYDRATION

- The Dry Eye Drink™ by Bruder®. Available in AM and PM formulations and three flavors.

Talk to Your Eyecare Professional
or order online at Bruder.com



Bruder
a Hico Vision Company

Distributed by Bruder Healthcare.
Alpharetta, GA 30004.
888-827-8337 | www.bruder.com
INST98900_2403



HEALTHY
EYES
START HERE™



iLidClean®
a Hico Vision Company

Silicone Eyelid Cleaning Tool

1 per pack

INSTRUCTIONS

- 1.** Cleanse your hands and your iLidClean® with a simple, unscented soap and water.
- 2.** Hold your iLidClean® near the last two to three rings at the base of the handle.
- 3.** Dip the tip of your iLidClean® in your preferred commercial or doctor-recommended eyelid cleaning product or makeup remover.
- 4.** With your eye closed, position the side of the iLidClean® nubbed cone along the eyelid margin at the base of the lashes. The long side of the cone should be along the lash line (not the eyelid).
- 5.** Using a gentle side-to-side motion for 10 seconds, use the long side of the nubbed cone to remove makeup, allergens, bacteria, oils and other "biofilm" from the eyelashes and the base of the lashes of the upper and lower eyelids. The corners of the eye or more precise targeted areas can be cleaned using either end of the tool (cone-end or blunt handle end). Take care to avoid the eye / corneal surface. It is normal for a few old and fragile eyelashes to drop when first using iLidClean®. Wash or wipe away the debris you removed using a warm, wet cloth.
- 6.** Rinse your iLidClean® and then run the long side of the nubbed cone along the length of the eyelashes (as if you were applying mascara) from the base to the tips of the lash.
- 7.** Last, rinse your iLidClean® again and use the nubbed tip to clean and massage the eyelid waterline (eyelid margin).
- 8.** Rinse the iLidClean® and repeat on the other eye.
- 9.** When done, wash your iLidClean® with a mild soap and water. Air dry and store in its specially designed case.

Hold Here



Clean Here



iLidClean®
flexes as
you clean

